

GRAF *Baby-Sono-Fix*



Precision and Safety



Sonofix Carrier-frame for hip-ultrasonography



Intended Use

Sono-Fix assures an exact hip-ultrasonic-testing for babies up to the age of one year. It is possible to fix the baby exactly in a lateral position for reaching precise ultrasonic scans.

It's a medical device of category I in terms of the European directive 93/42/EC in the valid version.

The carrier-frame, consisting of high-quality materials, enables a quick and gentle examination without outside help.

Placement:



DANGER

The Sono-Fix may only be placed on flat, horizontal and dry (non-slippery) surfaces!

The Sono-Fix may only be used with the carrier-frame, which is properly fixed on the base plate (picture 6: correct mounted fixing-strap).

All materials are washable (see chapter "cleaning", page 3). Due to hygiene factors the Sono-Fix should only be used with a proper cover-drapery/nappy (if necessary aseptic). The used materials and those surfaces respectively are NON-ASEPTIC.

Please also consider the material data sheet of the used cellular material (page 5).




DANGER


Never leave the baby lying unattended inside the Sono-Fix!

Buildup

The Sono-Fix consists of:

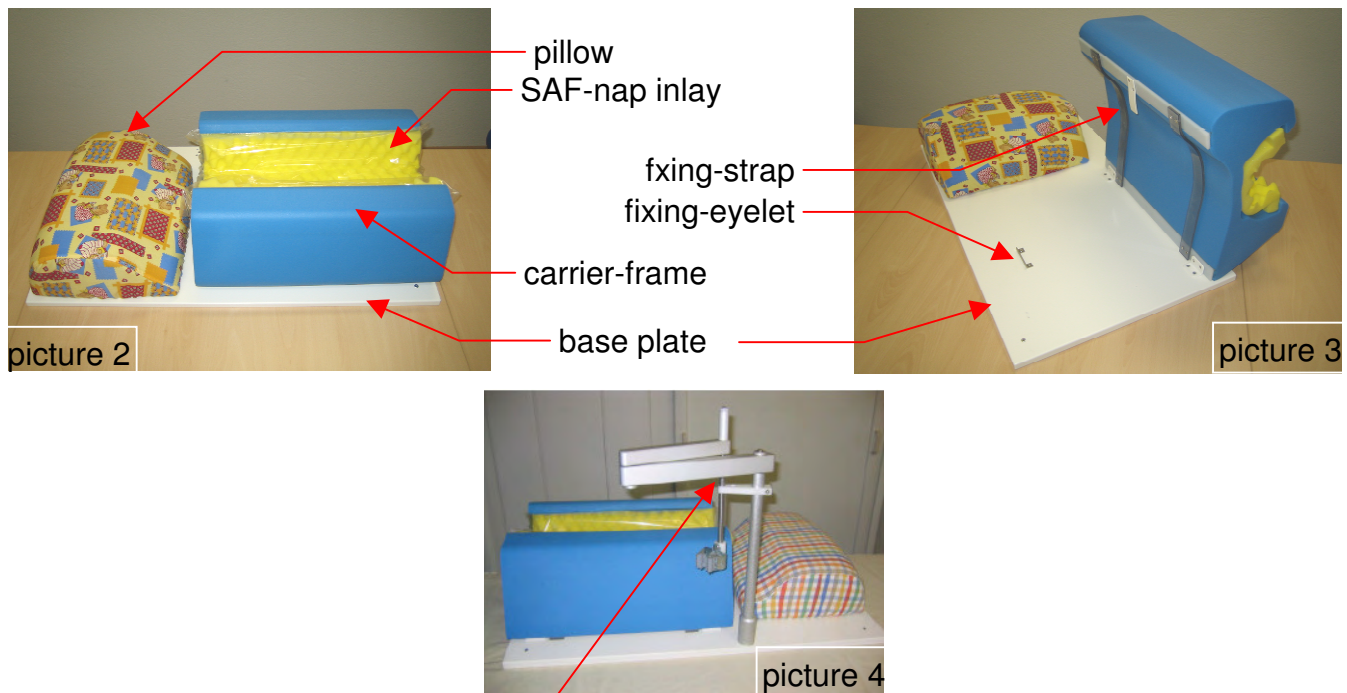

For an improved ultrasonic sensor movement a Sonoguide can be used optionally.

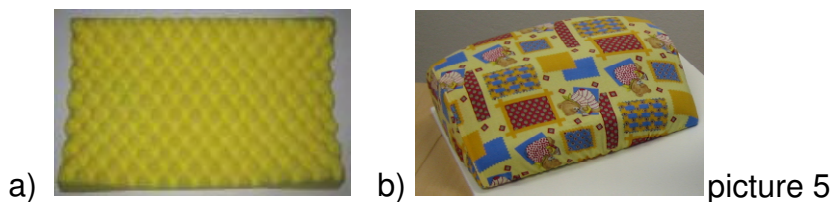

The SAF-nap inlay (visco-elastic material) and the pillow-cover can also be delivered individually.

Cleaning

Parts of the cellular material and the base plate:

To clean the Sono-Fix the carrier-frame can be flapped up (picture 2). For this step the fixing-strap must be pulled out of the fixing-eyelet (picture 6).

The cleaning is carried out with disinfectant (formalin-dilution, light-duty detergent, etc.) and by a temperature of max. 50 °C.



Pillow-cover

The pillow-cover can be removed easily from the pillow. It's pre-laundered and can be washed with light-duty detergent and a temperature of 30 °C.

Storage

The Graf Baby-Sono-Fix has to be stored in a dry place and under prevention of direct insolation.

Marking

The Sono-Fix is marked with following information on the back of the base-plate:


Serial - Number:

XXXXXXXXXX

Production date:

DDMMYY

GRAF - *Baby-Sono-Fix*

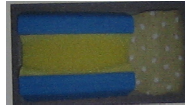

Medical device of cat. I
NON-ASEPTIC

Tapezierer- & Decorateur- Meisterbetrieb

Gebr. HIRSCHBECK GmbH

8843 St. Peter am Kammerberg 161
AUSTRIA

Phone.: +43 664 5238930

Fax.: +43 3536 8235 14

e-mail: info@hirschbeck.eu

www.hirschbeck.eu

Made in Austria